## User Information

### POWDER FREE SYNTHETIC STRETCH GLOVES

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#### 1 - EU Type-Examination

a - This product is classed as Category I Personal Protective Equipment (PPE) according to PPE Regulation (EU) 2016/425 and has been shown to comply with this Regulation through the Harmonised European Standards EN 420:2003+A1:2009

b - The EU Declaration of Conformity is accessible at [www.mediprotec.com](http://www.mediprotec.com)

#### 2 - EN 420:2003+A1:2009

Clause 5.1 - Length and fit : PASS

Clause 5.2 - Dexterity : Level 5

Glove size	7 - S	8 - M	9 - L	10 - XL
Minimum length / mm	230	240	250	260

Performance level	1	2	3	4	5
Diameter of dexterity pin / mm	11.0	9.5	8.0	6.5	5.0

#### Gloves for Special Applications (EN 420:2003, Clause 5.1.3)

These gloves are manufactured with unique standard sleeve length. Therefore, they are not meeting the Clause 5.1.3 from rule EN420:2003, describing the sleeves length for each size of gloves. Nevertheless, as special application for these gloves is to protect fingers and palm area from chemical splashes, they can be considered as totally adapted to their special use.

#### 3 - Product Instruction for Use

**a - Usage** - For Single Use only. If re-used, the risk of contamination and infection increases due to improper cleaning processes; and increased risk of holes and tear during re-use to weakening of gloves by cleaning processes.

**b - Sizing** - Select the right size glove for your hand

**c - Donning** - Hold glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to get a good fit. Don the other glove by the same procedure.

**d - Inspection** - Punctures or tears may occur after donning. Inspect each glove after donning, and immediately discontinue use if found damaged.

**e - Doffing** - Hold glove bead and pull toward the fingers until glove comes off.

**f - Disposal** - Properly disposal of all used gloves. Follow your Institution's policies for disposal.

**g** - The gloves were found to meet with the REACH annex XVII requirements for Polycyclic Aromatic Hydrocarbons (PAHs).

**h** - Components used in glove manufacturing may cause allergic reactions in some users. If allergic reactions occur, seek for medical advice immediately.

#### 4 - Handling and Storage

Store in a cool and dry place. Opened boxes should be kept away from fluorescent and sunlight. Gloves are packed in dispenser which is suitable for transport. Keep the gloves in the box when not in use.

#### 5 - Shelf life

The shelf life of product is 5 years from date of manufacture.

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